EarWell™

Infant Ear Correction System

Parent Instructions for use and care

The contained information will assist in maximizing the successful results from your child's EarWell™ course of therapy.

Please read all information and carefully follow all instructions.

U.S. and International patents pending

Becon Medical Limited
4640 East Sunrise Drive
Tucson, AZ 85718
520-881-0480
866-431-0480
Fax: 520-881-0941
www.beconmedical.com

© Copyright Becon Medical Limited
48BM0008Rev.B

Your child's physician has applied the EarWell™ ear correction system. At the conclusion of the therapy your baby's ears should be properly formed.

During the course of therapy there are several things that you should know to enhance the results of the EarWell™ application.

ADHESIVE

- EarWell™ is affixed to your baby's skin by a special medical adhesive designed to maximize adhesion and minimize trauma. It will hold the EarWell™ reliably in place for approximately fourteen to seventeen days.
- Inspect the cradle daily to verify its continued adhesive skin bond. When you notice that the cradle adhesive begins to separate from the skin, call your physician to schedule an appointment. During the appointment a new set of EarWell™ cradles will be re-applied.
- After phoning your physician for an appointment, use the provided roll of tape to temporarily secure the EarWell™ to the skin. Cut a small piece of tape from the roll (3 inches or less) and apply over the top of the EarWell™ and then onto the skin of your child. If the area of separation is larger than the width of the tape, use additional pieces of tape to secure it to the skin.
- There are also occasions when one or both EarWell™ cradles prematurely separate from the skin. This can occur when a child becomes unusually active or restless resulting in vigorous movement of arms and shoulders. Upon observing such separation use the tape to secure the cradle to the skin in the manner described above.
- Contact your physician upon securing the EarWell™ with tape.
- If the EarWell™ becomes completely dislodged, or for more than 50% of the EarWell™ surface area, DO NOT attempt to tape it back in place. Immediately call your physician to schedule an appointment.

- It is important to note that the EarWell™ becoming temporarily dislodged will not affect the results or the duration of the therapy.

- Simply schedule an appointment for as soon as possible.

WASHING

- Because adhesives are an intricate part of the EarWell™ attachment, it should remain as dry as possible.
- It is recommended that you do not wet the EarWell™ or attempt to wash it.

REDNESS

- EarWell™ is made of a clear material to permit ear visibility.
- If the ear becomes unusually red in color, report it to the applying physician.

CRADLE MAINTENANCE

- Care should be given to keeping the cradle interior clear of milk, formula or vomit. If any of these fluids penetrate to the inside of the cradle, make an appointment with the applying physician for removal of the top shell and cleaning of the ear and the EarWell™ components.
- Particular attention should be devoted to protecting the cradle interior from the presence of any vomit.

COVER REMOVAL

- The top cover of the EarWell™ should only be removed by a physician.

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